



medi

# Eclipse<sup>®</sup>

Super Absorbent Dressings

Effective Exudate  
Management

*Adv*ancis  
Medical

medi. I feel better.

## Five layers of confidence for effective exudate management



### Backing layer

- Non-strikethrough, preventing the patient embarrassment associated with leaks
- High moisture vapour transfer rate (MVTR)
- Water-resistant
- Protection from bacteria and infection



### Secondary backing layer

- Helps retain the dressing's shape, encouraging even distribution of exudate



### CrystalLock layer

- Absorbs exudate and locks it within the dressing to prevent leakage
- Associated bacteria is locked away too, which reduces the risk of infection



### Rapid wicking layer

- Draws exudate from the wound bed
- Ensures the wound doesn't dry but maceration is avoided
- Exudate is delivered into the CrystalLock layer



### Soft silicone contact layer

- Gently adheres to skin
- Will not adhere to wound bed



## Features

- Highly absorbent
- Soft and conformable
- Can be used safely under compression
- Market leading range of sizes
- Unique atraumatic wound contact layer

## Benefits

- Reduces the risk of maceration
- Secure fit for patient comfort
- Enhanced healing rates
- Absorption without bulkiness
- Minimize trauma and pain on removal

### Eclipse®

#### Description

Eclipse® is a super absorbent dressing with Crystalock® technology and non-strikethrough backing.

#### Indications

Moderate to heavily exuding wounds that require compression or retention bandaging.



How much fluid does Eclipse hold?\*\*\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
<b>Eclipse®</b>	CR4503	3in x 3in	10	A6196	69
Super absorbent dressing	CR3818	4in x 4in	20	A6196	122
	CR3769	6in x 6in	20	A6197	275
	CR4439	8in x 8in	10	A6197	488
	CR3743	8in x 12in	20	A6198	732
	CR3808	24in x 16in	10	A6198	2928

### Eclipse® ADHERENT

#### Description

Eclipse® Adherent is a super absorbent dressing with Crystalock® technology, non-strikethrough backing and soft silicone contact layer.

#### Indications

Moderate to heavily exuding wounds that benefit from soft-adherence and require compression or retention bandaging.



How much fluid does Eclipse hold?\*\*\*



Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
<b>Eclipse® Adherent</b>	CR4504	3in x 3in	10	A6196	69
Super absorbent dressing with silicone contact layer	CR3881	4in x 4in	10	A6196	122
	CR3883	4in x 8in	10	A6197	244
	CR3863	6in x 6in	10	A6197	275
	CR3864	8in x 12in	10	A6198	732

### Eclipse® BORDER

#### Description

Eclipse® Border and Eclipse® Border Oval are super absorbent dressings with Crystalock® technology, non-strikethrough backing, soft silicone contact layer and border, designed to be self-adhesive.

#### Indications

Moderate to heavily exuding wounds suitable for a bordered dressing.



How much fluid does Eclipse hold?\*\*\*



Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
<b>Eclipse® Border</b>					
Super absorbent dressing with silicone contact layer and border	CR4523	5in x 5in	10	A6196	191
	CR4296	6in x 6in	10	A6197	275
	CR4297	8in x 12in	10	A6198	732
<b>Eclipse® Border Oval</b>					
Super absorbent dressing with silicone contact layer and border	CR4300	4in x 8in	10	A6197	244
	CR4299	6in x 8in	10	A6197	366
	CR4298	8in x 12in	10	A6197	732

\*\*\*All figures are estimated based on absorbency testing report TR1397 result of 121.91g/100cm<sup>2</sup>.

### Eclipse® FOOT

#### Description

Eclipse® Foot is a super absorbent dressing with Crystallock® technology and non-strikethrough backing. It is uniquely shaped to fit securely around the foot, which is often difficult to dress with regular shaped dressings.

#### Indications

Moderate to heavily exuding wounds on the foot.



How much fluid does Eclipse hold? \*\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
---------	-----	-------	------------	-------------	----

#### Eclipse® Foot

Super absorbent foot wrap dressing

CR4219

13in x 19in

5

A6198

1932

#### Application:



### Eclipse® BOOT

#### Description

Eclipse® Boot is a super absorbent dressing with Crystallock® technology and non-strikethrough backing. It is uniquely shaped to fit securely around the lower leg.

#### Indications

Moderate to heavily exuding wounds on the lower leg.



How much fluid does Eclipse hold? \*\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
---------	-----	-------	------------	-------------	----

#### Eclipse® Boot

Super absorbent leg wrap dressing

CR4237

22in x 19in (SM)

5

A6198

3154

CR4014

24in x 28in (LG)

5

A6198

5124

CR4229

28in x 31in (XL)

5

A6198

5680

#### Eclipse® Boot Measuring Guide

Item #	Ankle	Calf
--------	-------	------

#### Small (CR4237)

7in to 8.5in

10in to 15in

#### Large (CR4014)

8.5in to 12in

15in to 21in

#### Xlarge (CR4229)

12in to 14in

19.5in to 26in






# Eclipse® Boot Application Guide

1



**Open out the dressing fully, white side up placing the heel into the small triangle, the top of the triangle pointing towards the patient**

2

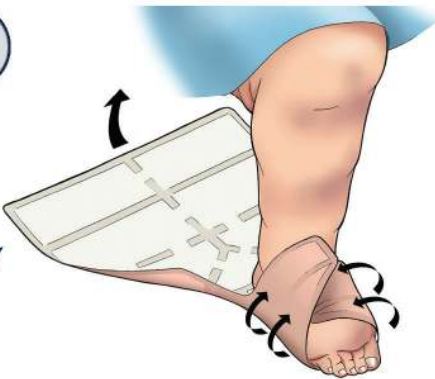

**Fold over the foot one at a time the smaller sections to the left and right and secure with tape**

3



**Fold both the larger sections one at a time around the leg also securing with tape**

## *Tips:*

- Eclipse® Boot can be used under compression
- Wear time will depend on the level of exudate, daily changes may be required, but Eclipse® Boot can be left in place for up to seven days
- Do not cut through the super absorbent pockets on Eclipse® Boot however Eclipse® Boot can be cut along the welded edges to fit a shorter leg or area

# Eclipse®

## CONTOUR

### Description

Eclipse® Contour is a super absorbent dressing with Crystallock® technology and non-strikethrough backing. It is uniquely shaped to mold to body contours ensuring optimum contact with the wound.

### Indications

Moderate to heavily exuding wounds in difficult to dress areas like the underarm, abdomen, back, lower leg and thigh.



How much fluid does Eclipse hold?\*\*\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
---------	-----	-------	------------	-------------	----

### Eclipse® Contour

Super absorbent multisite dressing

CR4398

12in x 20in

10

A6198

1867

# Eclipse®

## ADHERENT SACRAL

### Description

Eclipse® Adherent Sacral is a super absorbent dressing with Crystallock® technology, non-strikethrough backing and soft silicone contact layer, shaped to fit the sacral area.

### Indications

Moderate to heavily exuding wounds on the sacral area.



How much fluid does Eclipse hold?\*\*\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
---------	-----	-------	------------	-------------	----

### Eclipse®

#### Adherent Sacral

Super absorbent sacral dressing with silicone contact layer

CR3985

6.5in x 7.5in

10

A6197

394

CR3986

8.5in x 9in

10

A6198

617

6 \*\*\*All figures are estimated based on absorbency testing report TR1397 result of 121.91g/100cm<sup>2</sup>.

# Eclipse® Frequently Asked Questions

## Can Eclipse® be used as both a primary and secondary dressing?

Eclipse® can be used on all wound types as either a primary or secondary dressing.

## What type of wounds is Eclipse® indicated for?

Eclipse is indicated for all wound types with moderate to heavy drainage.

## How long can you leave Eclipse® on for?

Wear time will depend on the level of exudate. Daily changes may be required, but Eclipse® can be left in place for up to 7 days.

## Can Eclipse® be cut?

All Eclipse® dressings cannot be cut, the edges are sealed, cutting would lead to shedding of the internal components of the dressing.

## What should I use to secure Eclipse®?

Eclipse® and Eclipse Adherent® should be secured with the appropriate bandage system, i.e. tubular or compression. The Eclipse Border® dressings are self-adherent due to the soft silicone contact layer and the adhesive on the border, although additional securing can be used as mentioned above.

## Can Eclipse® be used under compression?

Yes Eclipse® can be used under compression. Eclipse® is soft and conforming and has the least effect on sub-bandage pressures when polymers in the dressing expanded.

## Can Eclipse® be used under pneumatic compression systems?

Yes, the thin and conforming design of the Eclipse is ideal for capturing fluid while patients are utilizing pneumatic compression therapy.

## Are there any contraindications?

Not advised for use on arterial bleeds or bleeding wounds, making it difficult to assess blood loss. Not advised for individuals with a known silicone sensitivity.

# Ordering information

# Eclipse®

Super absorbant, high-capacity dressing to manage high exudate applications

## Eclipse® Super absorbent dressings

How much fluid does Eclipse hold?\*

Product	SKU	Sizes	Pack Sizes	HCPCS* Code	mL
<b>Eclipse®</b> Super absorbent dressing	CR4503	3in x 3in	10	A6196	69
	CR3818	4in x 4in	20	A6196	122
	CR3769	6in x 6in	20	A6197	275
	CR4439	8in x 8in	10	A6197	488
	CR3743	8in x 12in	20	A6198	732
	CR3808	24in x 16in	10	A6198	2928
<b>Eclipse® Adherent</b> Super absorbent dressing with silicone contact layer	CR4504	3in x 3in	10	A6196	69
	CR3881	4in x 4in	10	A6196	122
	CR3883	4in x 8in	10	A6197	244
	CR3863	6in x 6in	10	A6197	275
	CR3864	8in x 12in	10	A6198	732
<b>Eclipse® Border</b> Super absorbent dressing with silicone contact layer and border	CR4523	5in x 5in	10	A6196	191
	CR4296	6in x 6in	10	A6197	275
	CR4297	8in x 12in	10	A6198	732
<b>Eclipse® Border Oval</b> Super absorbent dressing with silicone contact layer and border	CR4300	4in x 8in	10	A6197	244
	CR4299	6in x 8in	10	A6197	366
	CR4298	8in x 12in	10	A6197	732
<b>Eclipse® Foot</b> Super absorbent foot wrap dressing	CR4219	13in x 19in	5	A6198	1932
<b>Eclipse® Boot</b> Super absorbent leg wrap dressing	CR4237	22in x 19in (SM)	5	A6198	3154
	CR4014	24in x 28in (LG)	5	A6198	5124
	CR4229	28in x 31in (XL)	5	A6198	6930
<b>Eclipse® Contour</b> Super absorbent multisite dressing	CR4398	12in x 20in	10	A6198	1867
<b>Eclipse® Adherent Sacral</b> Super absorbent sacral dressing with silicone contact layer	CR3985	6.5in x 7.5in	10	A6197	394
	CR3986	8.5in x 9in	10	A6198	617

**XL Boot holds 1 gallon!**

\*Code usage should be verified by responsible parties in accordance with all published guidelines. Eclipse dressings are typically billed as:

**Alginate or other fiber gelling dressing**

\*\*All figures are estimated based on absorbency testing report TR1397 result of 121.91g/100cm<sup>2</sup>.